## NL - NEDERLANDS

### Gebrek van Lexie als warmtekussen:

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## GB - ENGLISH

### Using the Lexie:

- Warm/Cold pillow for the body.
- Warm: Entspannt und lindert Bauchschmerzen während der Menstruation.
- Nack- und Schulterwärme.
- Pain relief for lower back pain.
- Relaxation: the warmth and weight will help you relax.
- During pregnancy: as a heat pad for the relief of ligament pain and lower back pain (LBP).
- As a heat pillow: provides soothing heat to relieve tension, stress and pain.
- During pregnancy: as a heat pad for the relief of ligament pain and lower back pain (LBP).
- Ideal as a cold compress for muscle aches, swelling, insect bites, sunburn, headaches, fever, summer heat and hot flushes.
- As a heat pillow: provides soothing heat to relieve tension, stress and pain.
- It is suitable for a wide range of uses and conforms well to the neck, shoulders, back, head, feet and abdomen.
- The Lexie is fragrance-free.

### WARNING:

- Always follow these instructions and keep them in a safe place. Always read the instructions before use.
- Always use with the cover
  - The pillow should not be used by anyone who is hypersensitive or insensitive to heat and other very vulnerable people who are unable to respond to overheating.
  - Do not heat the pillow if it has come from another source.
  - Never use the pillow if it does not get wet! The pillow can no longer be used.
  - To ensure a long service life, it is very important to set the microwave correctly when it is heated.
  - Not recommended for children under 4 years of age or for use on individuals with medical conditions, who are disabled, asept or insensitive to heat (those unable to respond to overheating).
  - Always use under supervision.
  - The pillow should only be used on a covered part of the body.
  - The pillow should only be used on a covered part of the body.
  - Do not heat the pillow in a regular oven or a convection oven as those methods of heating are more difficult to control.
  - Do not let the pillow come into contact with metal while it is being heated. For example, touch the grill, the wall or the bottom of the microwave. This will damage the pillow.
  - Always check the temperature of the pillow with the inside of your wrist before use.
  - Do not use if there are any signs of skin irritation.

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- WARNING:**

- Always follow these instructions and keep them in a safe place. Always read the instructions before use.

- Always use with the cover

- The pillow should not be used by anyone who is hypersensitive or insensitive to heat and other very vulnerable people who are unable to respond to overheating.

- Do not heat the pillow if it has come from another source.

- Never use the pillow if it does not get wet! The pillow can no longer be used.

- To ensure a long service life, it is very important to set the microwave correctly when it is heated.

- Not recommended for children under 4 years of age or for use on individuals with medical conditions, who are disabled, asept or insensitive to heat (those unable to respond to overheating).

- Always use under supervision.

- The pillow should only be used on a covered part of the body.

- The pillow should only be used on a covered part of the body.

- Do not heat the pillow in a regular oven or a convection oven as those methods of heating are more difficult to control.

- Do not let the pillow come into contact with metal while it is being heated. For example, touch the grill, the wall or the bottom of the microwave. This will damage the pillow.

- Always check the temperature of the pillow with the inside of your wrist before use.

- Do not use if there are any signs of skin irritation.

- Do not use if there are any symptoms of inflammation.
- The pillow should not be washed.
- Do not use the pillow if it is worn or has overheated (the seeds will char and give off an unpleasant baking smell, or the pillow cover will show scorch marks).
- The cotton fabric will be affected and irreparably damaged if the padding gets too hot and the seeds retain too much heat. This can usually be seen by the brown spots and small holes that appear in the fabric. The pillow is then unusable. Moreover, the burnt smell is also unpleasant.
- Caution: Do not use the pillow while heating it in an oven or microwave. There is a risk of fire if the heating process is not carried out properly.
- KipKep cannot be held responsible for any damage caused by using the Lexie improperly or by failing to follow the instructions for using the pillow.
- Any serious incident that occurs when using the pillow should be reported to the manufacturer (KipKep B.V.) and the competent authority of the country where the user/patient resides.
- Always follow your microwave manufacturer's instructions to be sure of the power level for each setting on your microwave.
- Composition, filling: lined | Pillow: 100% cotton | Cover: varies, see label in cover. Always use the pillow with the original KipKep cover.
- Check the pillow regularly for signs of wear or damage. If there are any signs that the pillow has been used incorrectly or is not working, contact the supplier before using it again.

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